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1. Visualiser les alertes

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The screenshot displays a web interface for managing alerts. At the top, there is a navigation bar with a home icon, a settings gear, a user profile icon, and a hamburger menu icon. The main content area is divided into two sections. The first section, labeled '1', shows a table of alerts not taken care of. It has five columns: 'Prénom Nom', 'Feu Jaune (non remplissage)', 'Feu Jaune', 'Feu Orange', and 'Feu Rouge'. The first row is for 'anvers' with a red bell icon, a blue 'X' in the first 'Feu Jaune' column, a blue 'X' in the second 'Feu Jaune' column, a blue 'X' in the 'Feu Orange' column, and a red exclamation mark in the 'Feu Rouge' column. The second row is for 'John Smith' with a red bell icon, a blue 'X' in the first 'Feu Jaune' column, a yellow exclamation mark in the second 'Feu Jaune' column, a blue 'X' in the 'Feu Orange' column, and a blue 'X' in the 'Feu Rouge' column. The second section, labeled '2', is titled 'Alertes prises en charge' and shows '5 utilisateurs'. It has the same column structure. The first row is for 'Harry Covert' with a red bell icon, a blue 'X' in the first 'Feu Jaune' column, a yellow exclamation mark in the second 'Feu Jaune' column, a red exclamation mark in the 'Feu Orange' column, and a red exclamation mark in the 'Feu Rouge' column. The second row is for 'convention' with a red bell icon, a blue 'X' in the first 'Feu Jaune' column, a blue 'X' in the second 'Feu Jaune' column, a blue 'X' in the 'Feu Orange' column, and a red exclamation mark in the 'Feu Rouge' column. Each row in both tables has a blue menu icon on the right.

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The screenshot shows the 'Feu Orange' alert management interface. At the top, there is a header with an orange exclamation mark icon, the text 'Feu Orange', and a timestamp 'générée le 19/03/2018 à 14:10'. Below the header, there are two buttons: 'Déclarer une action' (highlighted with a red box and a blue circle '1') and 'Alerte injustifiée'. A close button 'X' is also present. The 'Type de l'action' section (highlighted with a red box and a blue circle '2') contains seven icons: a telephone, an envelope, an ambulance, a hospital building, a person, a speech bubble, and a question mark. Below this is a text input field (highlighted with a red box and a blue circle '3') with the placeholder text 'Veuillez préciser les informations exhaustives et datées à propos de vos observations et agissements'. Underneath is a section for 'Qui est intervenu sur cette action ?' (highlighted with a red box) containing a dropdown menu for 'Profil', a text input field for 'Nom, prénom, fonction...', and a trash icon. Below this is a '+ Ajouter un profil' link. At the bottom, there is a 'Clôturer l'alerte ?' section (highlighted with a red box and a blue circle '4') with 'Oui' and 'Non' radio buttons. A final 'Ajouter l'action' button is at the very bottom.

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